


Pyxis ES Training Modules


Go to <https://bd.com/LearningCompass>

If you haven't already created an account, create an account by clicking on **New User Registration** to access the site for the Pyxis ES online training portal. Must allow popups.

A. Select **"Content Library"** at top of webpage.

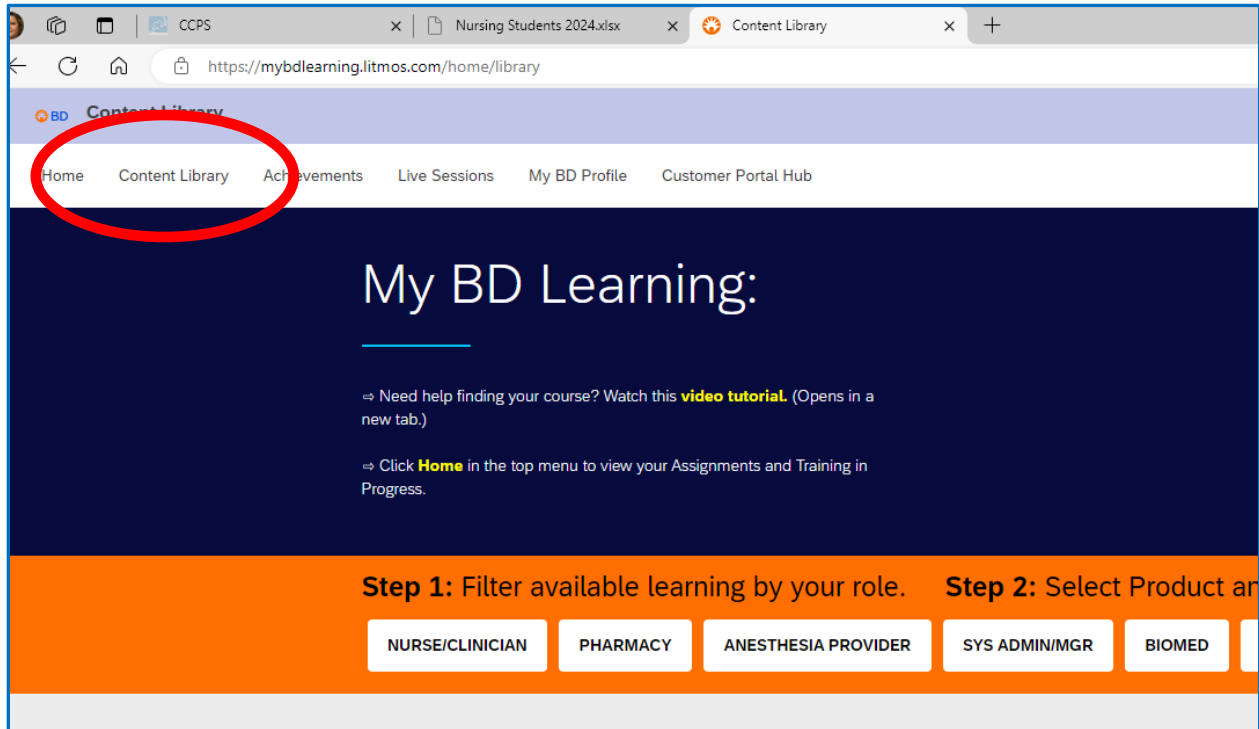


B. Select Nurse/Clinician to filter by your role. Under Topics section on the left side of webpage, type in **Pyxis ES**, then select box next to **Pyxis ES System v1.6**. The **Nurse/Clinician - Pyxis MedStation ES v1.6: Essentials Both Profile and Non-Profile Modes** course generates on the right side of the screen. Click on the Nurse/Clinician course.

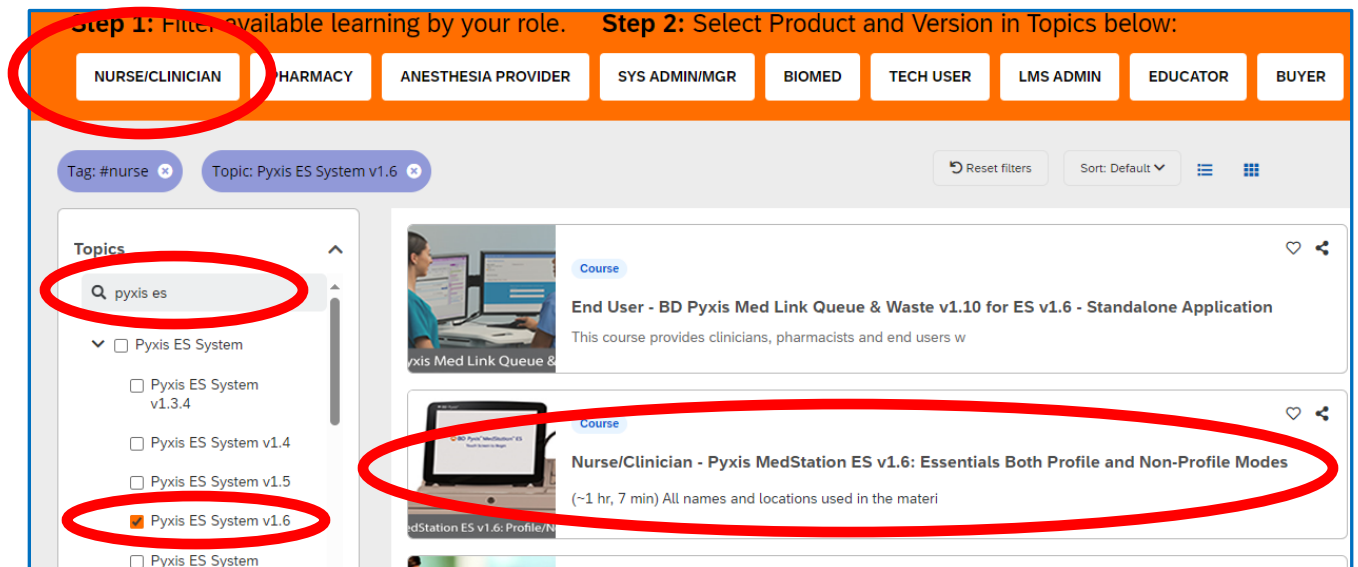

Pyxis ES Training Modules

C. Select **Start this course**. It will take approximately 1 hour and 7 minutes to complete all lessons.

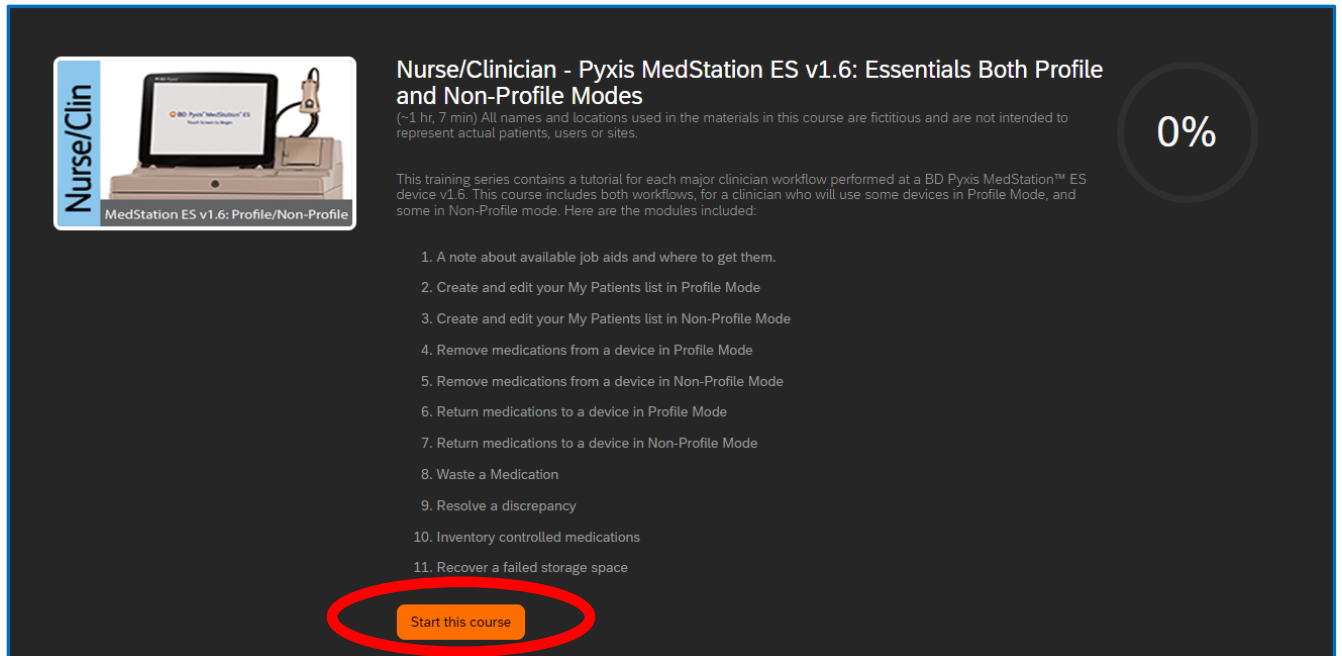

Nurse/Clin

Nurse/Clinician - Pyxis MedStation ES v1.6: Essentials Both Profile and Non-Profile Modes


(~1 hr, 7 min) All names and locations used in the materials in this course are fictitious and are not intended to represent actual patients, users or sites.

This training series contains a tutorial for each major clinician workflow performed at a BD Pyxis MedStation™ ES device v1.6. This course includes both workflows, for a clinician who will use some devices in Profile Mode, and some in Non-Profile mode. Here are the modules included:

1. A note about available job aids and where to get them.
2. Create and edit your My Patients list in Profile Mode
3. Create and edit your My Patients list in Non-Profile Mode
4. Remove medications from a device in Profile Mode
5. Remove medications from a device in Non-Profile Mode
6. Return medications to a device in Profile Mode
7. Return medications to a device in Non-Profile Mode
8. Waste a Medication
9. Resolve a discrepancy
10. Inventory controlled medications
11. Recover a failed storage space

Start this course

D. Select **Next Module** and the Available job aids will generate.

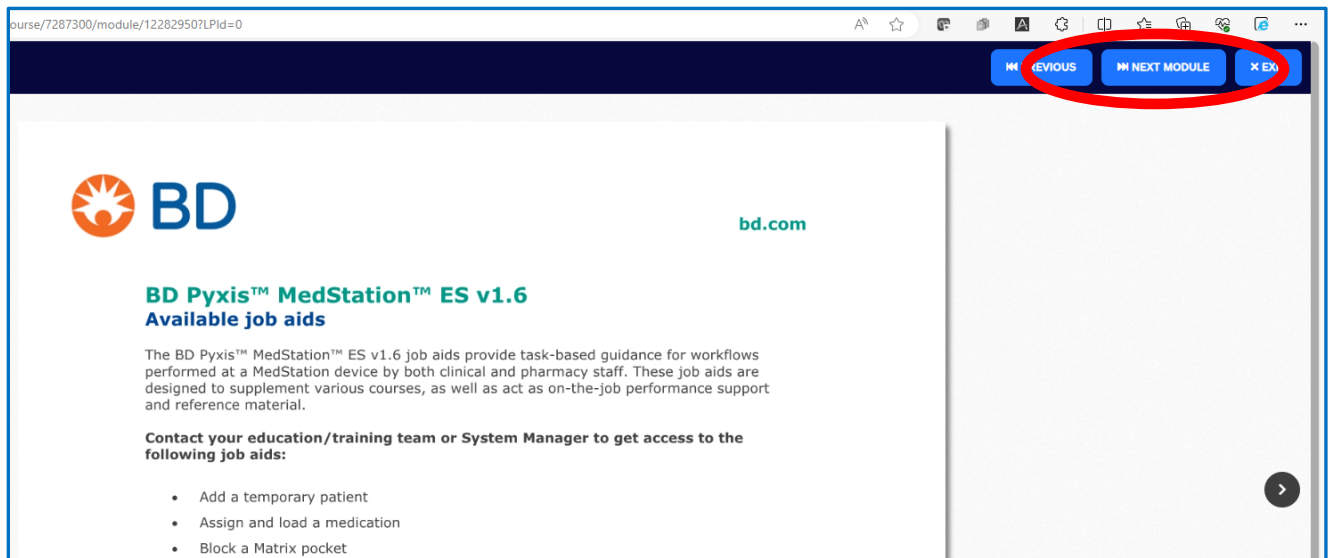

course/7287300/module/12282950?LPId=0

BD **bd.com**

BD Pyxis™ MedStation™ ES v1.6
Available job aids

The BD Pyxis™ MedStation™ ES v1.6 job aids provide task-based guidance for workflows performed at a MedStation device by both clinical and pharmacy staff. These job aids are designed to supplement various courses, as well as act as on-the-job performance support and reference material.

Contact your education/training team or System Manager to get access to the following job aids:

- Add a temporary patient
- Assign and load a medication
- Block a Matrix pocket

PREVIOUS **NEXT MODULE** **X EXIT**

Pyxis ES Training Modules

- E. Select **Next Module** upon completion (video and simulation) of a module to move through the lessons.

